DEPRESOR LINGUAL

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TONGUE DEPRESSOR

Only for single use. The reuse of this product may cause infections. Available in two materials:

- High quality wood, with shatterproof consistency and rounded edges to prevent injuries in the oral area.
- Non-toxic plastic (polystyrene), biocompatible and with rounded edges to prevent injuries in the oral area.

With an appropriate mechanical consistency to guarantee the impossibility of breakage during its use. Used as tongue depressor to get correct views of the oral area and to prevent contamination during throat or nasopharyngeal sampling. In bulk presentation.

The depressor is intended to be used in clinical and hospital environments. The use of the depressor presents no difficulty for the intended users. Not suitable for any other application than its intended use. Do not use if dirt is observed on the depressor. Discard the product if any broken or chipped depressor is found. Any strong pressure during use, although unlikely, could break the depressor. In case of any damage, discard immediately. Depressors do not need special storage conditions. Due to the intended use of the product, the end users belong to the health sector and they have biological waste containers, where the used products should be discarded and managed as biological waste. The depressors, once used, must be decontaminated, or treated as contaminated medical waste. It is the responsibility of each laboratory to handle, treat and dispose of waste according to the regulations. Unused depressors can be considered as non-hazardous products and can be discarded according to this criterion.

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